
Instruction leaflet

Coupling in two steps / Anschluss in 2 Schritten
Couplage en 2 étapes / Acoplamiento en 2 pasos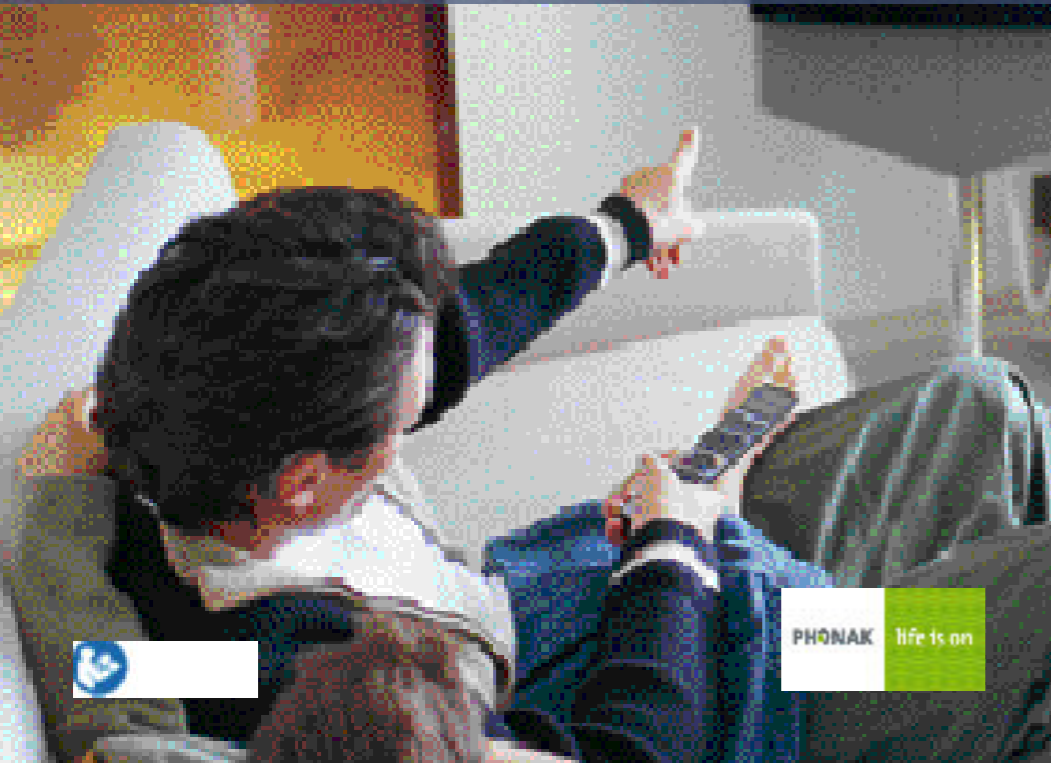

Pairing and connecting iCom with the Voiis Stereo – Wireless Music Gateway

1. Connecting the Voiis transmitter to your TV

- Select one of Cinch, SCART or the 3.5 mm jack plugs to connect your TV ①
- Plug the 3.5 mm jack into the "IN" port of the Voiis ②
- Connect the Voiis to the main plug
- Turn off all other Bluetooth enabled devices

2. Pairing iCom and Voiis transmitter

In order to receive TV-audio iCom must be wirelessly connected to the Voiis transmitter. This process is called pairing and is only needed one time when installing the devices for the first time.

On iCom

- Turn on your iCom ③
- Press the Bluetooth pairing button ④ for 2 seconds until the Bluetooth indicator ⑤ flashes blue
- Your iCom will remain in pairing mode for two minutes or until the pairing is completed

On the Voiis

- Press and hold the Voiis front button ⑥ for 3 seconds until the Bluetooth indicator ⑦ flashes blue
- Within 1 minute both Bluetooth indicators ⑤, ⑦ turn solid blue. This indicates the end of the pairing process

Switching Voiis transmitter ON/OFF

- To switch OFF, press the front button ⑥ twice until the light indicators below headset and speaker symbol ⑧, ⑨ are switched off
- To switch ON press the front button ⑥ until the yellow light below the headset symbol ⑧ turns on

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